

Tools required: an adjustable wrench

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the 69623A / 69663A Tub-Guard® Bathtub Safety Rail. If you do not understand any part of these warnings, cautions, or instructions, contact a healthcare professional for direction in the use of this product. If the Bathtub Safety Rail is not properly assembled and adjusted, personal injury and damage to the Bathtub Safety Rail could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Tub-Guard® Bathtub Safety Rail.**
- ⚠ **WARNING: This product has a weight limit of 250 lb (113 kg), EVENLY DISTRIBUTED.**
- ⚠ **WARNING: This product is NOT recommended for use on fiberglass or plastic bathtubs. GF Health Products, Inc. ("GF") assumes no responsibility for any damage or injury caused by improper installation or use of this product.**

### INSTALLATION

- ⚠ **WARNING: DO NOT overtighten Bathtub Safety Rail adjustment handle.**

1. Locate section of bathtub wall where Bathtub Safety Rail is to be mounted. The Bathtub Safety Rail should be mounted ONLY on the straight — not curved — part of bathtub wall. Ensure the tub surface where the Bathtub Safety Rail will be installed is smooth, clean, and dry before installing the Bathtub Safety Rail. Remove any dust, powder, or foreign particles.
2. Fold out the adjustment handle from the recess in Bathtub Safety Rail body.
3. Loosen the adjustment handle by turning it counter-clockwise until clamping mechanism is wide enough to fit over bathtub wall. Place Bathtub Safety Rail down onto bathtub wall. Adjustment handle should be outside the bathtub, and clamping mechanism inside the bathtub. Ensure the Bathtub Safety Rail is fully seated before tightening.
4. Tighten the adjustment handle by turning it clockwise until the Bathtub Safety Rail is firmly in place on bathtub wall. DO NOT overtighten the adjustment handle.
5. Once unit is firmly in place, fold down the adjustment handle until the handle locks back into the recess in the Bathtub Safety Rail body.
6. To test mounting before use, apply a strong push and pull motion to the Bathtub Safety Rail. If the Bathtub Safety Rail moves, reset it and tighten it more.

- ⚠ **WARNING: Ensure the Tub-Guard Bathtub Safety Rail is installed as described before use.**

### OPERATION


- ⚠ **WARNING: Always ensure the Bathtub Safety Rail is secure before use.**
- ⚠ **WARNING: Hold the Bathtub Safety Rail firmly when entering or exiting the bathtub.**

### MAINTENANCE / CLEANING

- ⚠ **WARNING: Check the Bathtub Safety Rail weekly for signs of damage or cracking. If damage or cracking of any component of the Bathtub Safety Rail is found, DO NOT use the Bathtub Safety Rail. Contact your GF authorized provider.**

Inspect the Tub-Guard® Bathtub Safety Rail weekly and tighten it as needed to ensure proper installation.

To clean the Bathtub Safety Rail, wash it with soap and water or a mild detergent, then rinse. Ensure the Bathtub Safety Rail is rinsed well before use.

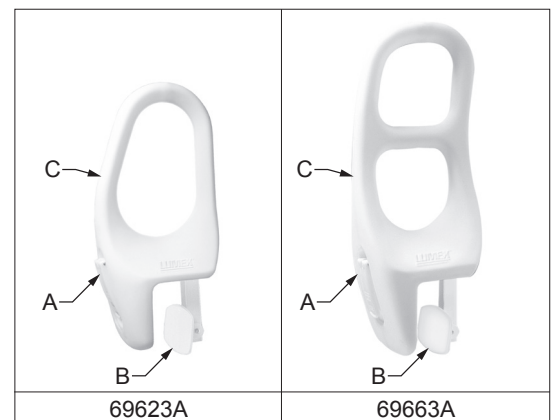

- A) adjustment handle
- B) clamping mechanism (one side only shown)
- C) safety rail body

**Tub-Guard Bathtub Safety Rails**

# LIMITED WARRANTY

## SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the Original Purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the Original Purchaser, to the extent permitted. Original Purchaser is one who purchases this product new and unused from GF or a Certified GF Dealer/Distributor.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

Bathtub Rail:.....3 years

\*Labor is not included in the warranty.

†The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the Original Purchaser, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

## OBTAINING WARRANTY SERVICE

Contact the Dealer/Distributor from whom the product was purchased. If there is not a Dealer/Distributor, you must contact GF directly by calling (770) 368-4700, sending a fax request to 770-368-4932 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). The Customer Service Representative will provide specific directions. Failure to abide by the specific directions will result in denial of the warranty claim.

## EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

## ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES AND COUNTRIES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE INSTANCES, GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

## NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: [www.grahamfield.com](http://www.grahamfield.com).
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**LUMEX®**  
by graham-field

Manufactured By:

GF Health Products, Inc.  
One Graham-Field Way, Atlanta, GA 30340  
Made in China



+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

graham  
field

© 2003, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

LX\_69623A-INS-LAB-RevD23