

- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using the Full-Body Sling. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Full-Body Sling is not properly applied, personal injury and damage to the Full-Body Sling could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your Graham-Field authorized distributor immediately.**
- ⚠ **WARNING: Carefully inspect the sling before each use for wear and damage to seams, fabric, straps, and strap loops. Torn, cut, frayed or broken slings can fail, resulting in serious personal injury to the user. Use only slings that are in good condition. Discard and destroy old, unusable slings.**


ITEM	Size	Maximum Weight Capacity, EVENLY DISTRIBUTED
FM110, F112, FMC114	Medium	450 lb
FM111, F113, FMC115	Large	450 lb
FMC116, F117	X Large	450 lb
FM140, FMC141	Heavy Duty, XX Large	600 lb

- ⚠ **WARNING: DO NOT exceed the Full-Body Sling's maximum weight capacity, shown in table above.**
- ⚠ **WARNING: Cancer and Reproductive Harm - [www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper application or use of this product.**

## APPLICATION


1. Check the patient's weight and the sling's maximum weight capacity. Ensure the patient's weight does not exceed the sling's maximum weight capacity.

- Review the following procedures carefully before attempting lift; call GF Health Products, Inc. Technical Support at 1.770.368.4700 with any questions.

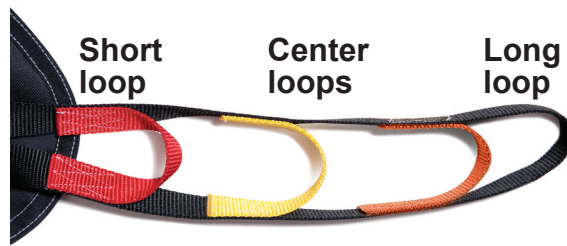

Sling Loop Options	
Long loop	reclined position
Center loops	semi-reclined position
Short loop	most upright position

- See the Sling Loop Options table above to determine the appropriate position.

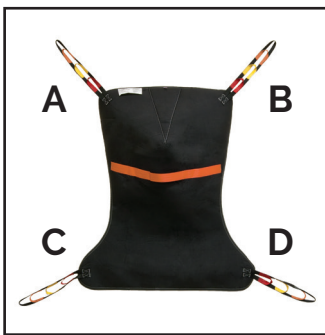

**Figure 1**



**Figure 2**



**Figure 3**



**Figure 4**

- See Figure 1 above for correct positioning of sling on spreader bar.
- Position the sling under and behind the patient's back with the color adjustment strap on outside. Drape **A** and **B** over the shoulders as shown in Figure 2 above.
- Carefully, with help from another caregiver, if needed, pull the sling portion **C** and **D** forward to the bend of the knee as shown in Figure 2 above.
- Hook sling loops **A** and **B** to spreader bar **a** and **b** as shown in Figure 3 above.
- Hook sling loops **C** and **D** to spreader bar **c** and **d** as shown in Figure 3 above.
- Do not lock caster brakes, in order to allow the lift to properly balance weight.
- Proceed with lift and transfer as shown in Figure 4 above.

## MAINTENANCE

To clean sling, use a mild soap and water solution and a clean cloth. Rinse with clear water, then allow to air dry.




**▲ NOTICE: DO NOT use bleach, solvents, abrasive cleaners, or scouring pads on any part of the sling.**

## FEATURES

Mesh Full-Body Commode Sling		
Quick-draining/drying mesh fabric offers full head and neck support. For use with 4-point spreader option. Designed to meet the requirements of HCPCS Code E0621.		
<b>FMC114</b>	Medium	1 ea
<b>FMC115</b>	Large	1 ea
<b>FMC116</b>	X Large	1 ea
<b>FMC141</b>	Heavy Duty, XX Large	1 ea

Mesh Full-Body Sling		
Quick-draining/drying mesh fabric offers full head and neck support. For use with 4-point spreader option. Designed to meet the requirements of HCPCS Code E0621.		
<b>FM110</b>	Medium	1 ea
<b>FM111</b>	Large	1 ea
<b>FM140</b>	Heavy Duty, XX Large	1 ea

Solid Full-Body Sling		
Fully-padded fabric offers a soft, yet support surface. Offers full head and neck support. For use with 4-point spreader option. Designed to meet the requirements of HCPCS Code E0621.		
<b>F112</b>	Medium	1 ea
<b>F113</b>	Large	1 ea
<b>F117</b>	X Large	1 ea

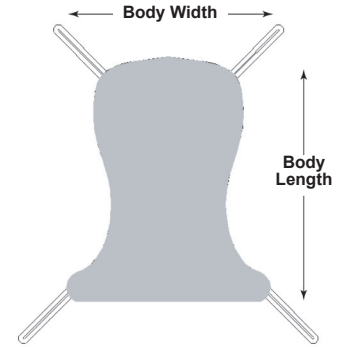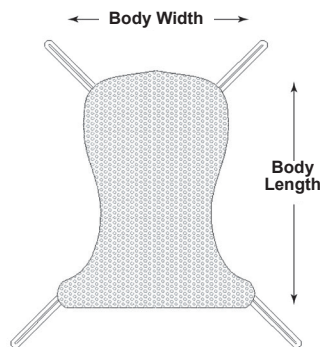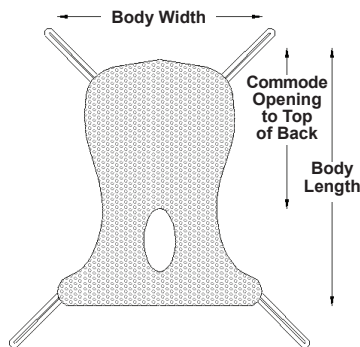

ITEM	Size	Body Width	Body Length	Commode Opening to Top of Back (FMC114, FMC115, FMC116, FMC141 only)	Maximum Weight Capacity, EVENLY DISTRIBUTED
<b>FM110, F112, FMC114</b>	Medium	42 3/4"	50 3/4"	34 3/4"	450 lb
<b>FM111, F113, FMC115</b>	Large	44 3/4"	54 1/4"	38"	450 lb
<b>FMC116, F117</b>	X Large	44 3/4"	60 1/4"	43"	450 lb
<b>FM140, FMC141</b>	Heavy Duty, XX Large	42"	51"	41 1/2"	600 lb

## WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**LUMEX**<sup>®</sup>  
by graham-field



Manufactured By:  
GF Health Products, Inc.  
One Graham-Field Way, Atlanta, GA 30340  
Made in USA

© 2013, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

LX\_F112-INS-LAB-RevD22



+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

**graham**  
**field**