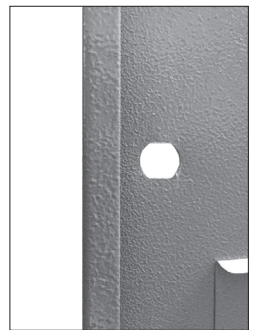


- ⚠ **WARNING: Important! Read and understand these instructions before installing the 3011 Inner Lock / 3012 Outer Lock. If the lock is not properly installed, personal injury and damage to the lock or Narcotic Safe may result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. Use only Grafco replacement parts. Non-Grafco replacement parts could cause personal injury and damage to the lock or Narcotic Safe.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**



**Figure 1**



**Figure 2**

**COMPONENTS**

Item	Description	Components Included
3011	Inner Lock, Narcotic Safe	1 Lock, 1 Locking Clip, and 2 Keys
3012	Outer Lock, Narcotic Safe	

**INSTALLATION (REFER TO PICTURES AT RIGHT)**

*Tools Required: Needle-Nose Pliers and Rubber or Wooden Mallet*

1. Release the Locking Clip with Needle-Nose Pliers as shown in Figure 1.
2. Remove the damaged lock.
3. Locate the new lock; unlock and remove the key.
4. Place the new lock in the punched hole, aligned as shown in Figure 3.
5. To secure the new lock, slide the locking clip with the mallet as shown in Figure 4.



**Figure 3**

**▲ NOTICE: Ensure lock is securely installed before using Narcotic Safe.**



**Figure 4**