

Intended Use: Even heating of ultrasound gel or lotion bottle.

⚠ IMPORTANT SAFETY INFORMATION: PLEASE READ

⚠ DANGER: To reduce the risk of electrocution:

Always unplug this product immediately after using.

Do not use this product while bathing.

Do not place or store this product where it can fall or be pulled into a tub or sink.

Do not place this product in, immerse in, or drop into water or other liquid.

Do not reach for a product that has fallen into water. Unplug immediately.

⚠ WARNING: To reduce the risk of burns, electrocution, fire, or personal injury:

Do not leave this product unattended when plugged in.

Close supervision is necessary when this product is used by, on, or near children or invalids.

Use this product only for its intended use as described in these instructions.

Do not use attachments not recommended by the manufacturer.

Do not operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water. Return the product to a service center for examination and repair.

Keep the cord away from heated surfaces.

Never use this product while sleeping or drowsy.

Never drop or insert any object into any opening or hose.

Do not use this product outdoors, operate where aerosol (spray) products are being used, or operate where oxygen is being administered.

Connect this product to a properly grounded outlet only.

If it is necessary to use an extension cord, use only a three-wire extension cord that has a three-blade grounding plug, and a three-slot receptacle that will accept the product's plug.

⚠ Important! Read and understand these instructions before assembling or using the GF108 / GW208 Gel / Lotion Warmer. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the Gel / Lotion Warmer is not properly assembled and operated, personal injury and / or damage to the Gel / Lotion Warmer could result.

If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Graham-Field replacement parts. Non-Graham-Field replacement parts could cause personal injury and / or damage to Gel / Lotion Warmer.


GF Health Products, Inc. ("Graham-Field") assumes no responsibility for any damage or injury caused by improper installation or use of this product.

INSTALLATION INSTRUCTIONS

The Gel / Lotion Warmer may be mounted to a wall or placed on a flat, stable surface. Plug the power cord into a power supply that matches the voltage listed on the electrical information label on the back of the unit.

Wall Mounting: The Gel / Lotion Warmer may be mounted on a wall using the keyhole slots on the back of the unit. Use mounting hardware capable of supporting a minimum of six pounds.

Storage: When storing Gel / Lotion Warmer, store it in a dry area. Coil the cord and secure.

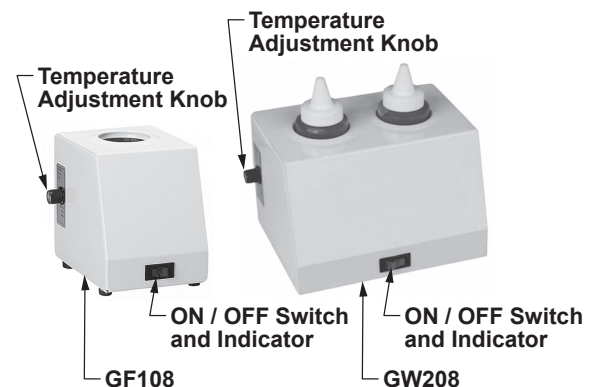

OPERATION INSTRUCTIONS

ON / OFF Indicator: The ON / OFF switch illuminates when the unit is operating.

Temperature Adjustment Knob: This knob allows adjustment of the operating temperature.

Setting the Operating Temperature: If a change in operating temperature is required, turn the knob clockwise to increase temperature; turn the knob counterclockwise to decrease temperature. Adjust the knob in small increments and allow sufficient time for the temperature to change.

Operating the Gel Warmer: Warm ultrasound gel only when needed. Turn the ON / OFF Switch ON. Allow sufficient time for the gel or lotion to reach optimum temperature — gels and lotions at room temperature may take up to 45 minutes to reach a comfortable temperature. When operation is complete, turn the ON / OFF Switch OFF. Remove bottles from the warmer as soon as possible and dry immediately. Do not store bottles in the warmer throughout the day.

⚠ WARNING: Test gel / lotion temperature before application to ensure comfortable temperature.

MAINTENANCE / CLEANING

⚠ WARNING: There are no serviceable components inside the Gel / Lotion Warmer. Do not open the outer casing.

Cleaning: Unplug the power cord before cleaning. Clean the Gel / Lotion Warmer weekly with a low-level hospital grade disinfectant. If the warmer becomes soiled, clean it immediately with a low-level hospital grade disinfectant. Do not submerge or immerse the unit in water.

Fuse: There is a thermal fuse located inside the unit. The fuse is not replaceable by the operator. If the Gel / Lotion Warmer does not operate when plugged into the correct voltage power supply, return the unit to your GF authorized distributor for replacement of the thermal fuse.

SPECIFICATIONS

Product temperature range:..... 90°F - 122°F

Environmental requirements:..... Indoor use, 70°F - 90°F

Supply voltage range: 120V, 60 Hz

Power rating:..... Model GF108 125W; Model GW208 250W

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

Ultrasound Gel.....one (1) year.

Warmer.....one (1) year.

Labor is not included in the warranty.

The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to cs@grahamfield.com. Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: www.grahamfield.com.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

Manufactured For:

GF Health Products, Inc.
One Graham-Field Way, Atlanta, GA 30340
Made in USA

© 2014, GF Health Products, Inc. All Rights Reserved.
Graham-Field is a trademark of GF Health Products, Inc.

GF108-INS-LAB-RevD24



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

graham
field