# LUMEX®

by graham-field

## Platinum Collection Bath Seat

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graham  
field

QTY. 1 EA

## SAFETY GUIDELINES - PLEASE READ BEFORE USE

- **DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Item has a maximum weight capacity of 350 lb - **EVENLY DISTRIBUTED**.
- GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- **ALWAYS** inspect the bath seat to ensure that it is properly positioned and stable before using. **DO NOT** use if bath seat is wobbly or unstable.
- If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. **DO NOT** use substitute parts.
- Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips **IMMEDIATELY**.
- **DO NOT** use this as a transfer bench or climbing device.
- To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping. Do not put downward weight on one side or corner of the Bath Seat. Do not put downward weight on the backrest. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.
- All four leg tips **MUST** be in contact with shower/tub floor at all times.
- Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes **BEFORE** use.
- Only use GF Health Products, Inc. accessories with this product.
- This product should only be used with tub floors wider than 18 ¾ inches.
- Users with limited physical capabilities must be supervised or assisted when using the bath seat.
- Make certain **ALL** screws and nuts are tight at all times.
- **DO NOT** submerge product in water, otherwise damage may result.

## MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## INSTALLING THE BACK (ONLY FOR 7921A-4)

1. Place bath seat on its side on a flat working surface.
2. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
3. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

**NOTE:** Ensure seat support tube buttons protrude fully through channel guide holes.

**CAUTION:** Backrest is not designed to support the full weight of the user.

## ADJUSTING LEG HEIGHT

Legs are already installed on the bath seat.

To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.

**NOTE:** An audible “click” will be heard.

3. Repeat STEPS 1 and 2 for the other three legs.

**NOTE:** Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.

## LIMITED WARRANTY

GF Health Products, Inc. (“GF”) warrants this product to be free from defects in materials and workmanship to the original retail purchaser or original lessee only (“Customer”) for:

**Frame:** five (5) years

**Tips, Wheels/Casters, and Handgrips:** three (3) months

The warranty does not cover: 1) other non-durable components; 2) defects, damage or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; and 3) products not installed, used, or properly cleaned and maintained in accordance with the applicable GF instructions. Product provided to a lessee must be inspected prior to delivery and must be maintained by the Dealer. The lessee must be properly trained on the operation and safety of this product prior to use.

GF will, at its option, repair or replace the defective product. Warranty issues must: 1) be reported to the Dealer from whom the Customer purchased the product within the applicable warranty period and 2) upon examination by GF or its authorized representative, prove to be a warranty item. GF is not responsible for freight or labor charges associated with the warranted item. If there is no Dealer, you must contact GF directly by calling 770-368-4700, faxing 770-368-4932 or e-mailing [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided.

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. GF’S TOTAL LIABILITY FOR THIS PRODUCT IS LIMITED TO THE COST OF THE PRODUCT(S) GIVING RISE TO THE CLAIM. IN NO EVENT WILL GF BE LIABLE FOR ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES. CERTAIN LAWS MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES FOR SPECIFIC PRODUCTS AND IN THOSE INSTANCES GF’S LIABILITY AND THE LIABILITY OF GF’S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW. This warranty gives you specific legal rights. You may also have other rights, which vary from state to state or country to country.

*For additional information related to warranty claims, please visit GF’s terms and conditions located on [www.grahamfield.com](http://www.grahamfield.com)*

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## Features

- Lightweight, durable, rust-resistant frame
- Wide seat with handles provides additional comfort
- Available with and without backrest
- Seat height adjusts in 1" increments for various user heights
- 350 lb weight capacity EVENLY DISTRIBUTED

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at [www.grahamfield.com](http://www.grahamfield.com)

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© 2009, GF Health Products, Inc.,  
One Graham-Field Way, Atlanta, Georgia 30340.  
Made in China. [www.grahamfield.com](http://www.grahamfield.com)