

- ⚠ **WARNING:** Important! Read and understand these instructions before installing the casters. If the casters are not properly installed, personal injury and damage to the chair could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. authorized distributor immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and cause damage to the casters or chair.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

Tools Required: M6 open-ended wrench and Phillips-head screwdriver.

HOW TO REPLACE COMMODORE CASTER(S)

Collect necessary tools and replacement caster(s).

1. To remove the old caster(s), use screwdriver and wrench to loosen screw and gasket at base of caster. Once loosened, directly pull caster from frame.
2. Insert new caster wheel assembly into frame tube.
3. Align holes of the wheel with holes of frame and insert screw through hole.
4. Place gasket at end of screw.
5. Use screwdriver and wrench to tighten caster into leg until completely secure.
6. Repeat with any remaining casters as necessary.

⚠ **CAUTION:** It is important to always check your casters for proper performance and maintain them on a regular basis.

⚠ **CAUTION:** It is important to always check your casters and all components of the chair for proper function before every use.



6810A-CLA



6810A-CSA