

- ⚠ **WARNING:** Important! Read and understand these instructions before installing the casters. If the casters are not properly installed, personal injury and damage to the Rollator could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. authorized distributor immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and cause damage to the wheels or Rollator.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

Tools Required: Allen key, wrench & Flat-head screwdriver.

HOW TO REPLACE FRONT WHEEL(S)

Collect necessary tools and replacement wheel(s).

1. To remove the old wheel(s), use a flat-head screwdriver to remove the black cover tips over the bolt.
2. Locate the nut and hold the Allen key firm in one hand in the first nut (*image 1*), while using the wrench to loosen wheel bearing.
3. Remove bolt and remove old wheel.
4. Reverse instructions for installation of the replacement wheel(s).

- ⚠ **CAUTION:** It is important to always check your wheels for proper performance and maintain them on a regular basis.
- ⚠ **CAUTION:** It is important to always check your wheels and all components of the rollator for proper function before every use.

HOW TO REPLACE REAR WHEEL(S)

Collect necessary tools and replacement wheel(s).

1. To remove the old wheel(s), use a flat-head screwdriver to remove the black cover tips from wheel bearings.
2. Using two wrenches for both sides, loosen bolts until wheel slides out easily.
3. Insert new wheel(s).
4. Using two wrenches, tighten bolt until wheel is secure.
5. Make sure black tips are replaced onto wheel bearings.
6. Discard any old parts.

- ⚠ **CAUTION:** It is important to always check your wheels for proper performance and maintain them on a regular basis.
- ⚠ **CAUTION:** It is important to always check your wheels and all components of the rollator for proper function before every use.



RJ4700



Front Wheels



#1



Rear Wheels