
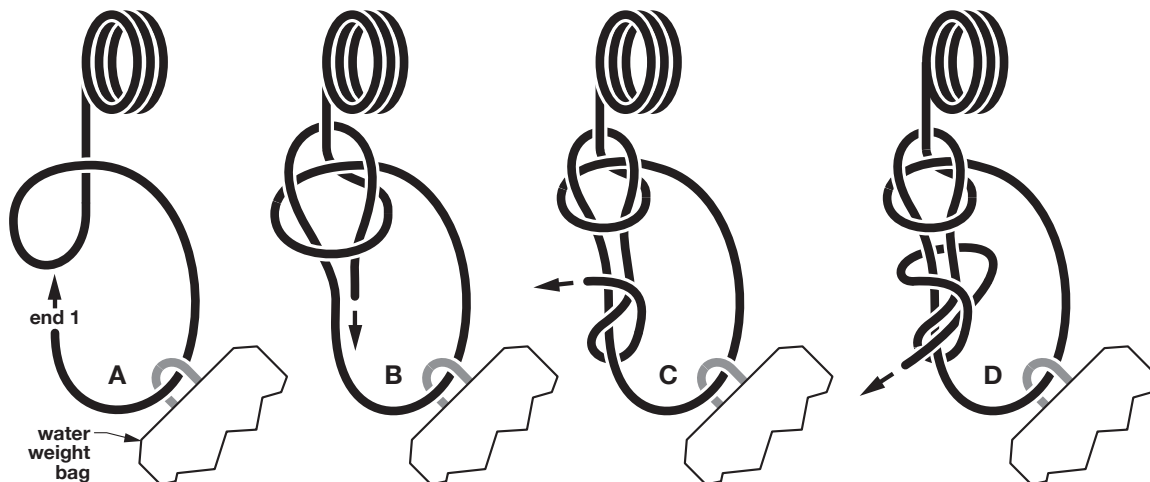


- ⚠ **WARNING:** Important! Use the GF1871 Cervical Traction Set only under the care of a physician. Read and understand these instructions before assembling or using the GF1871 Cervical Traction Set. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the GF1871 Cervical Traction Set is not properly assembled and adjusted, personal injury and/or damage to the Cervical Traction Set could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and/or damage to the Cervical Traction Set.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.



ASSEMBLY INSTRUCTIONS

Unpack box; remove components from bags. During assembly, refer to pictures throughout instructions to identify components.



1. Insert top of pulley hanger into slot in bracket; turn hanger until it fits into slot. After insertion, ensure that bump traps pulley hanger in slot as shown at right.
2. Ensure that pulleys are oriented as shown at right (bumps keep pulleys separated). Insert bottom of hanger pulley through bottom bracket lip as shown (this will require a little force).
3. Install bracket and pulley hanger over door as shown at right.
4. Insert one of the rope ends (end 1 in the picture below) through the ring at the top of the water weight bag; tie a secure knot, such as a bowline with a back-up knot, to fasten it. If you do not know how to tie a bowline with back-up knot, follow steps A-D below:

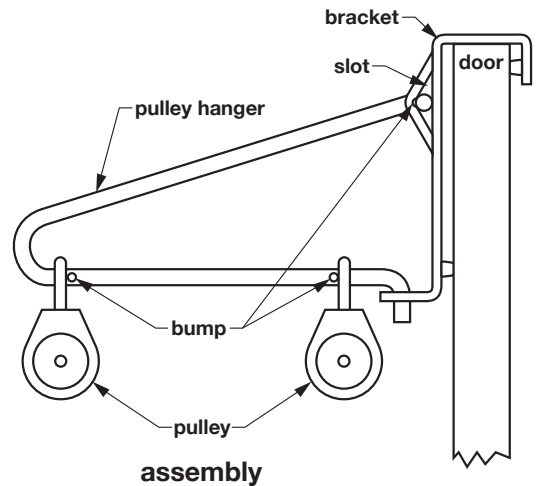

5. Pull rope ends taut. Before continuing, ensure that rope is taut and tied as instructed in step 4, with water weight bag securely installed on rope.
6. Fill water weight bag with physician-prescribed amount of water.

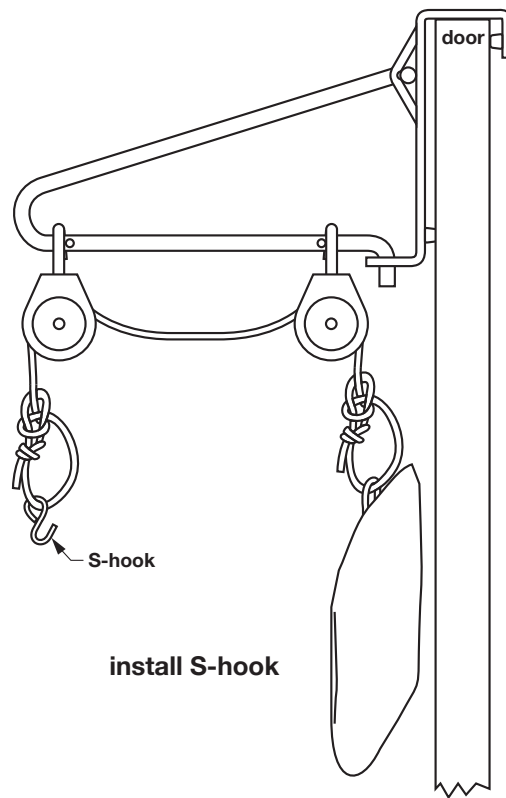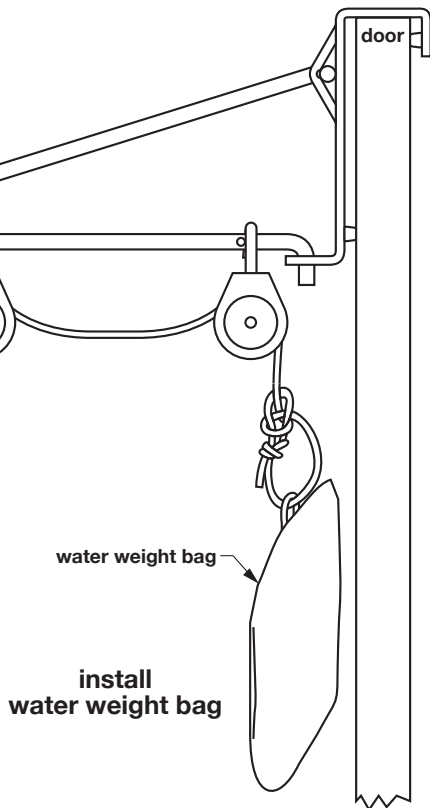

7. Thread free end of rope through pulleys with water weight bag at closest point to door, as shown at above left.
8. Install the closed loop of the S-hook on the free rope end and tie a secure knot, as you did in step 4. Pull rope ends taut. Before continuing, ensure that rope is taut and tied as instructed, with S-hook securely installed on rope, as shown at above right.

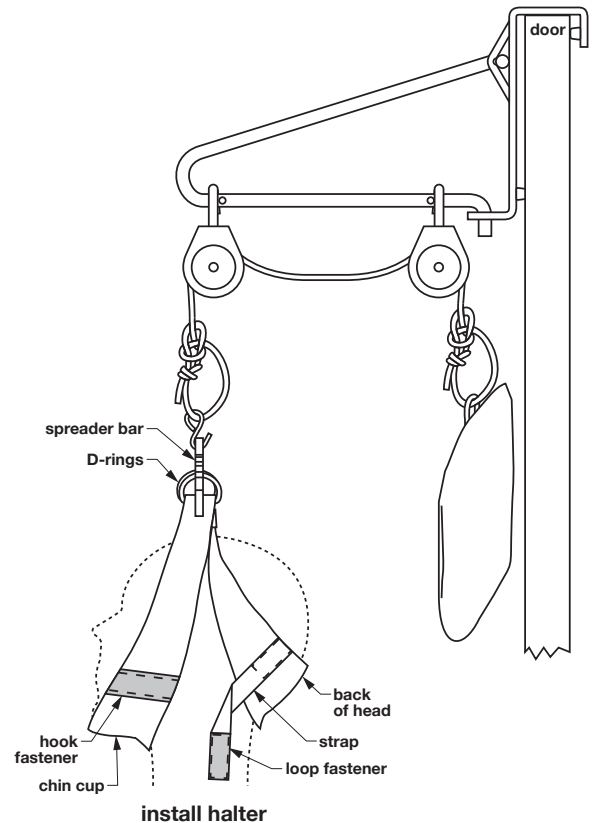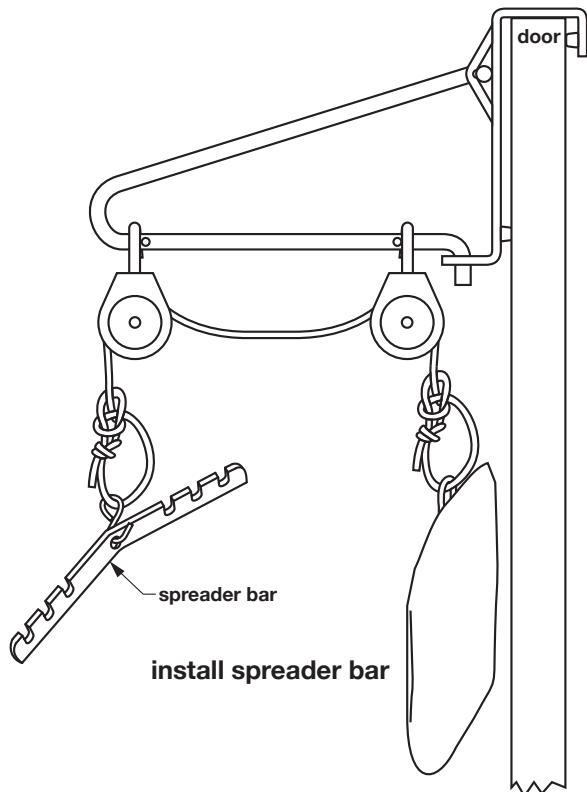


9. Hang spreader bar on S-hook as shown on previous page left image.
10. Install halter by D-rings on spreader bar slots as shown at above right, with straps and hook and loop fasteners oriented as shown, in the position recommended by your healthcare provider.

OPERATION INSTRUCTIONS

⚠ WARNING: Before each use, ensure knots are tight, all hardware is secure, and rope is not frayed. Ensure rope remains aligned on top of pulleys and moves freely; rope may get off-track and bind between pulley wheel and casing.

1. Sit in chair and face away from door as shown at upper right (or as your physician prescribes). Slip halter, oriented as explained in step 10 above, onto your head.
2. Fasten hook and loop fasteners as shown at right. Ensure halter fits comfortably; adjust fit by moving halter straps up or down at D-rings. Avoid any sudden, jerky movement.
3. Use the Cervical Traction Set for the prescribed amount of time; do not move during this time.
4. At end of prescribed time, lift water weight bag to relieve tension on halter. Detach hook and loop fasteners, and carefully remove halter from your head.

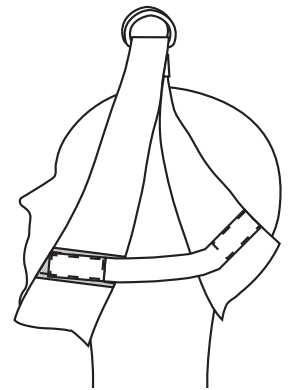

**attach hook and
loop fasteners**

CLEANING AND MAINTENANCE

To clean, hand wash rope and halter gently, using mild soap, rinse with clear water, and air dry. Other components: use a clean, soft cloth and a mild soap and water solution to clean, rinse with clear water, and air dry or dry with a clean, soft cloth.

▲ NOTICE: DO NOT use harsh cleansers or bleach to clean Cervical Traction Set.

ONE-YEAR LIMITED WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

Cervical Traction Set 1 (one) year

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to cs@grahamfield.com. Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

1. Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
2. Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
3. Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
4. Accessories or parts not provided by GF;
5. Matching of color, grain or texture except to commercially acceptable standards;
6. Changes in color caused by natural or artificial light;
7. Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
8. Any labor or shipping charges incurred in the replacement part installation or repair;
9. Costs and expenses of regular maintenance and cleaning; and
10. Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

1. Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: www.grahamfield.com.
2. Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
3. Claims for any short shipment must be made within three (3) days of the invoice date.



Manufactured For:
GF Health Products, Inc.
1 Graham-Field Way, Atlanta, GA 30340
Made in China

© 2007, GF Health Products, Inc. All Rights Reserved.
Lumex is a registered trademark of GF Health Products, Inc.

LX_GF1871-INS-LAB-RevG25



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

