

**SAVE THESE INSTRUCTIONS FOR FUTURE USE**

- ⚠ **WARNING: Important: Read and understand these instructions before assembling or using the PVC hamper. Ensure that the PVC hamper is installed as described in the following paragraphs before use. If the PVC hamper is not properly installed, personal injury and/or property damage could result.**
- ⚠ **WARNING: GF Health Products, Inc. ("Graham-Field") assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**
- ⚠ **WARNING: If components are damaged or missing, contact your Graham-Field dealer immediately. DO NOT use substitute parts.**

**HAMPER ASSEMBLY - REFER TO PICTURES AT RIGHT**

*Tools needed: Phillips screwdriver or cordless drill*



1. Remove partially assembled hamper from shipping container / box. Box should contain the following items:
  - a. Partially assembled hamper
  - b. Four caster wheels (two locking, two non-locking) in brown box
  - c. Lid handle (one for single unit; two for double unit)
  - d. Appropriate mesh/vinyl retainer bag/s
  - e. Appropriate wire bag holder/s for optional liner/s

**Note:** *The appropriate lid is attached to each hamper frame. To fit properly, each lid must be used on its own frame. DO NOT MIX lids on double Hampers.*

Double units must be assembled before continuing: Connect the fittings to the PVC extensions. Install 1/2" screws to fasten them. Use Phillips screwdriver to tighten.

2. Ensure that double units are assembled as described above. Install the four casters in the openings of the four-way fittings: Each caster fitting has a pre-drilled pilot hole on the inside. Secure the casters to the fittings with 1/2" screws. Use a Phillips screwdriver or cordless drill to tighten. Ensure that casters are securely fastened to fittings, or casters will fall out.
3. Secure the lid/s to the paddle/s. Use the nylon screws and bolts to fasten. Hand-tighten. Ensure that lid and paddle are securely fastened, or the hamper lid will not pivot properly.
4. Attach handle/s to lid/s: Install handle on top of lid. Insert screw through lid and into handle. Use a Phillips screwdriver or cordless drill to tighten.

- ⚠ **WARNING: Before use, ensure that hamper is correctly assembled as described and that all components and hardware are securely fastened.**

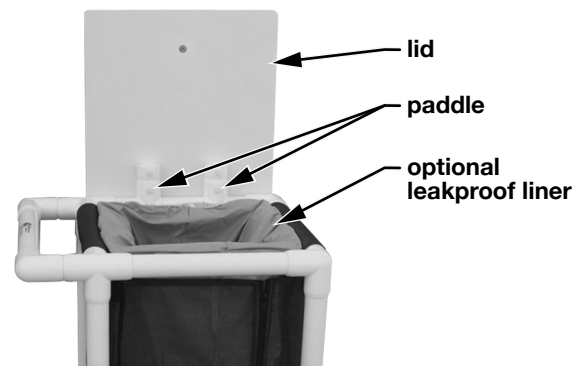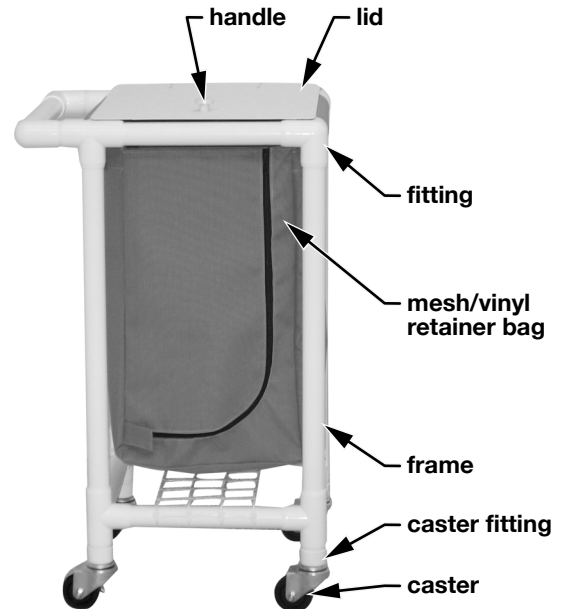

## WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item.

(See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

PVC frame:	1 (one) year
Mesh bag:	6 (six) months
Solid vinyl components:	6 (six) months

\* Labor is not included in the warranty.

† Upholstery is only warranted on material supplied by GF.

‡ The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: [www.grahamfield.com](http://www.grahamfield.com).
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

**LUMEX®**  
by graham-field

Manufactured For:  
GF Health Products, Inc.  
1 Graham-Field Way, Atlanta, GA 30340  
Made in U.S.A.

© 2007, GF Health Products, Inc. All Rights Reserved.  
Lumex is a registered trademark of GF Health Products, Inc.

GF0700174-INS-LAB-RevB25



+1 770.368.4700

Information contained herein is subject to change without notice.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)

**graham**  
**field**