

⚠ WARNING: The 8005 model has a maximum weight capacity of 450 lb (204 kg), **EVENLY DISTRIBUTED** and the 9005 model has a maximum weight capacity of 600 lb (272 kg), **EVENLY DISTRIBUTED**.

CONTENTS:

One (1) Partially assembled shower bed
Six (6) caster wheels (4 locking, 2 non-locking)
Push / Pull handles
Close Cell Mattress
Ten (10) 1/2" stainless steel screws



ASSEMBLY INSTRUCTIONS:

Step 1

Installing Casters:

1. Install the four locking casters into the female fittings on the base of the head and foot sections of the shower bed. Secure each caster to the fitting with one (1) 1/2" stainless steel screw.
2. Install the two non-locking casters to the female fittings on the base in the center of the frame. Secure with stainless steel screws.

⚠ WARNING: **ALWAYS** install the casters to bed before use.

Step 2

Install one push / pull handle to each bed end. Secure with 1/2" stainless steel screws.

▲ NOTICE: Safety rails can be held in place by one safety locking pin. Close cell mattress serves as a buffer / comfort for patient when using shower bed.

INSTRUCTIONS FOR USE:

⚠ WARNING: Before use, ensure the equipment has been properly assembled.

⚠ WARNING: The improper use of the bed or failure to comply with all directions and warnings may result in death or injury.

⚠ WARNING: Before use, ensure casters are all functioning properly and secured to the bed.

The main purpose for the shower bed is to bathe residents/patients that are confined to a bed. The easy rolling casters allow the shower bed to transfer bed ridden residents/patients to and from the shower room.

1. Roll the shower bed next to the individual's bed.
2. Drop the side rail facing the bed down, keeping the opposite side rail up.
3. If it's possible and the resident/patient has no medical requirements, transfer the resident/patient with head elevation in down/flat position.
4. Have a minimum of 2 persons (nurses, aids, etc.) transfer the resident/patient onto the shower bed.
5. Once the resident/patient is safely transferred and in a comfortable position, the head elevation can be raised if needed. Both side rails must be up and locked prior to the shower bed being moved.
6. When using the shower bed, direct the drain hose to a drain system. All excess water must drain properly.

⚠ WARNING: Caregiver should be present and alert at all times while bed is in use.

⚠ WARNING: Assistance should always be provided when transferring in and out of the bed.

⚠ WARNING: Bed may not be appropriate for all individuals. Assessment should be conducted by a skilled caregiver for proper suitability for the individual using the bed.

⚠ WARNING: The bed should only be used under the supervision of a trained and licensed healthcare provider and is intended to be used in a standard indoor environment.

⚠ WARNING: **ALWAYS** abide by the weight limit capacities for each product.

⚠ WARNING: **NEVER** allow the user to lean outside the bed frame.

⚠ WARNING: **NEVER** allow the user to suddenly shift weight in a any way creating a tipping hazard for the user and bed.

⚠ WARNING: **ALWAYS** use the bed's safety belt and / or shoulder harness (when provided).

- ⚠ **WARNING:** After any adjustments, repairs, or service make sure the bed has been re-assembled properly before use by individual.
- ⚠ **WARNING:** ALWAYS apply the brakes on casters when needed.
- ⚠ **WARNING:** DO NOT use this bed on an incline, always make sure all casters are in constant contact with the flooring surface.
- ⚠ **WARNING:** ALWAYS store bed in a dry ventilated environment.
- ⚠ **WARNING:** DO NOT Use bed is and of the following is evident:
- Bed appears unsteady or wobbly
 - Casters fail to move with ease
 - Fabric appears torn or sagging
 - Cracks are observed in the fittings or frame
 - Bed appears to be compromised in any way, shape, or form
- ⚠ **WARNING:** DO NOT attempt to modify the equipment, use only the casters and accessories provided for the bed. Any use of third-party casters and / or accessories may pose a hazard to the user and will void the manufacture warranty.

CARE AND MAINTENANCE

FOR PVC PIPING AND CASTERS:

- ⚠ **WARNING:** DO NOT use solvents.
- ⚠ **WARNING:** Wipe down piping with non-abrasive cleaner.
- ▲ **NOTICE:** Restore luster with spray wax.
- ▲ **NOTICE:** Remove hair and mop thread build-up.
- ▲ **NOTICE:** If casters are used in a wet environment, keep lubricated with white lithium paste or similar lubricant.
- ▲ **NOTICE:** Casters are subject to normal wear and tear, yearly replacement is recommended.

FOR MESH FABRIC (SLING):

- ⚠ **WARNING:** If laundering is required, use only COLD water in large machine with regular detergent and one-cup bleach.
- ⚠ **WARNING:** DO NOT use softeners or solvents.
- ⚠ **WARNING:** Spin-dry only; machine heat will shrink mesh fabric.
- ▲ **NOTICE:** Hang damp fabric on frame and allow to air dry, returning to its original shape.
- ▲ **NOTICE:** Subject to normal wear and tear, yearly replacement is recommended.