

EXAM ROOM SUPPLIES



SILVER NITRATE APPLICATORS

- Applications: skin or mucous membrane cauterization, removing warts and granulated tissue
- 75% Silver Nitrate and 25% Potassium Nitrate
- Ingredient Net Weight per 6" Applicator tip:
 - 50 mg (37.5 mg Silver Nitrate, 12.5 mg Potassium Nitrate)
 - Silver Nitrate tips are soluble in all secretions
 - Control strength by dipping in sterile, distilled water
 - Rigid wooden applicator for precise placement
 - Convenient tube vial
- Made in USA, GF Health Products, Inc. Rhode Island Facility
- ISO 13485 Certified Facility



U.S. Federal Law prohibits dispensing without prescription.



For professional use only.

Distributors (USA) must have an active state wholesale drug license to purchase Silver Nitrate Applicators.



Contraindications

Tissue staining: Silver salts can stain tissues black; this typically fades within two weeks.

Argyria risk: Prolonged exposure can cause argyria, a slate-blue discoloration of the skin and mucous membranes due to silver deposition.

Eye staining: The sclera (white part of the eye) may also be affected.



Warnings and Precautions

Poisonous if ingested: Can cause severe symptoms like gastroenteritis, coma, convulsions, and respiratory issues. Seek medical help immediately if poisoning occurs.

Eye contact danger: Do not use on eyes. If contact occurs, flush with water for 15 minutes and consult a physician.

Chemical burns: Silver nitrate is caustic and can cause burns if misused.



Usage Guidelines

Use **chemical-resistant gloves** and appropriate protective gear.

Confine application to the treatment area to avoid staining or burns.

Minimize contact time, especially on delicate skin or neonates.

Avoid prolonged contact with skin or surfaces to prevent staining.



1590



1592



1593

Item	NDC N°	Size		Active Ingredients /applicator	Double Dipped	UOM
		inch	cm	mg		
1590	12165-100-01	6.00	15.24	50	■	100/tube, 12 tubes/box
1592	12165-200-03	12.00	30.48	70	■	100/tube
1593	12165-300-01	18.00	45.72	98	■	1/ea

■ Denotes Standard Feature — Not Available / Does Not Apply



GF Health Products, Inc. is an
intertek ISO 13485:2016 / MDSAP Certified Company

Color variations may occur due to monitor or printer settings.

+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.

www.grahamfield.com



© 2007, GF Health Products, Inc. All Rights Reserved.
Graham-Field is a trademark of GF Health Products, Inc.

graham
field

GF0700173 Rev H25