

- ⚠ **DEALER:** These instructions **MUST** be provided to the user.
- ⚠ **WARNING:** Important! Read and understand these instructions before using FP3076 High Fall Pad. If the High Fall Pad is not properly used, personal injury and damage to the item could result. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- ⚠ **WARNING:** The High Fall Pad may create a trip hazard when positioned alongside the bed. Ensure anyone near or around the bed is aware of the pad's location to prevent tripping / falling.
- ⚠ **WARNING:** Children and pets should be supervised around the product to avoid accidental tripping.
- ⚠ **WARNING: NOT FOR SLEEPING:** This product is **NOT** designed or intended for sleeping or prolonged rest. Misuse may result in discomfort or injury.
- ⚠ **WARNING:** The FP3076 High Fall Pad has a maximum weight capacity of 500 lb (226.8 kg), **EVENLY DISTRIBUTED.**
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

CLEANING INSTRUCTIONS

- ⚠ **WARNING:** High Fall Pad **MUST** be cleaned regularly after each use.
- ⚠ **CAUTION:** Because of the potential risk of infectious exposure, cleaning with the patient on the bed is **NOT** recommended.
- ⚠ **WARNING:** All equipment should be inspected. Any item that is visibly soiled with the patient's blood or other body fluids should be properly cleaned or removed.
- ⚠ **WARNING:** Staff members should treat soiled High Fall Pad as if it were contaminated with pathogenic microorganisms.
- ⚠ **WARNING:** Staff members should wear appropriate protective clothing when cleaning the High Fall Pad.
- ⚠ **WARNING:** All cleaning solution(s) **MUST** be properly diluted according to the manufacturers' instructions.
- ⚠ **WARNING:** **DO NOT** open the High Fall Pad cover during cleaning, infection control procedures, or during use.
- ⚠ **CAUTION:** **DO NOT** autoclave or launder High Fall Pad cover.
- ⚠ **CAUTION:** To avoid fabric breakdown and loss of fluid resistance, use only these types of properly diluted standard cleaning solutions: Quaternary Wipe Down, Phenolic Wipe Down, and 8-10% Bleach / Water Wipe down.

ROUTINE CARE:

Wipe clean with neutral soap suds and warm water. Rinse with clean water and allow to air dry.

- ⚠ **CAUTION:** Follow standard institutional wipe down and infection control procedures.

HARD TO CLEAN AREAS:

Use standard liquid cleaner and a damp sponge. Harsh solvents, cleaners or detergents should not be used when cleaning the High Fall Pad cover.

DISINFECTION:

Germicides and disinfectants should be diluted as specified on product label by manufacturer. A 1/10 Bleach / Water solution may also be used to wipe down High Fall Pad cover and allowed to air dry.

INTENDED USE

The High Fall Pad is intended to be used with the Basic American Amerilux bed **ONLY**. The pad is approximately the same height as the bed frame in its lowest position together with a standard 6" (15.4 cm) mattress. The High Fall Pad is intended to be positioned on the floor alongside the bed frame to help reduce the risk of a bed falls.

UNPACKING

1. Check for obvious damage to the carton or its contents. If damage is evident, notify the carrier and your Graham-Field authorized distributor.
2. Remove all packaging materials.

INSTRUCTION FOR USE

WARNING: Improper placement of the High Fall Pad can cause serious injury.


1. Unclasp the High Fall Pad and unfold.
2. Lower the Amerilux bed all the way to the floor.
3. Place the High Fall Pad next to the bed ensuring the pad is flush against the outer most part of the bed

NOTE: DO NOT place pad under the bed frame or accessories such as assist devices or sideboards.

⚠ **WARNING: Allow enough space around the High Fall Pad so that resident is able to egress off pad.**

⚠ **WARNING: The High Fall Pad may create a trip hazard when positioned alongside the bed. Ensure anyone near or around the bed is aware of the pad's location to prevent tripping / falling.**

Important Properly placed High Fall Pad:

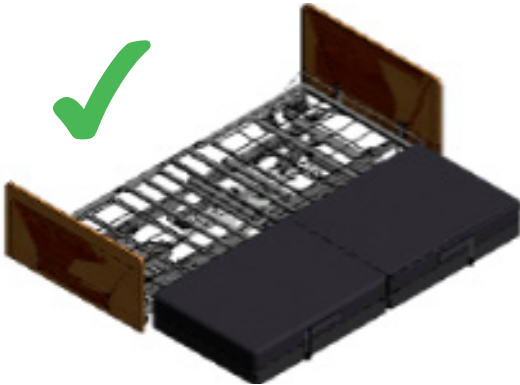

NOTE: The High Fall Pad can be folded in half for easier storage while not in use.

Important Incorrect placement:



⚠ **WARNING: DO NOT place the High Fall Pad under the bed during use. Interference may cause uneven lifting of the bed and risk of patient injury and product damage.**

⚠ **WARNING: DO NOT put fall pad flush with casters.**

⚠ **WARNING: DO NOT store the High Fall Pad under the bed. Accidental lowering of the bed onto the pad can cause instability and result in patient injury and product damage.**

IT IS A CONDITION OF THIS WARRANTY THAT THE FOLLOWING ARE COMPLIED WITH:

1. All instructions for cleaning and maintenance of the Fall Pad are adhered to;
2. The internal and external parts of any product are inspected at least once every three (3) months for any damage and wear. All damaged parts are replaced immediately, taken out of service until the replacement part has been obtained, and the damaged part has been replaced.

LIMITED WARRANTY
SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the Original Purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the Original Purchaser, to the extent permitted. Original Purchaser is one who purchases this product new and unused from GF or a Certified GF Dealer/Distributor.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time period are set forth below:

- Foam:.....five (5) years
- Upholstered Cover:two (2) years

* Labor is not included in the warranty.

‡ The warranty period is as designated above. If a part is replaced under warranty, the original warranty period will not be affected. All other replacement parts will be subject to the warranty period specified.

The applicable warranty period shall commence from date of shipment to the Original Purchaser, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

Contact the Dealer/Distributor from whom the product was purchased. If there is not a Dealer/Distributor, you must contact GF directly by calling (770) 368-4700, or by e-mailing a request to cs@grahamfield.com. The Customer Service Representative will provide specific directions. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

1. Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
2. Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
3. Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
4. Accessories or parts not provided by GF;
5. Matching of color, grain or texture except to commercially acceptable standards;
6. Changes in color caused by natural or artificial light;
7. Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
8. Any labor or shipping charges incurred in the replacement part installation or repair;
9. Costs and expenses of regular maintenance and cleaning; and
10. Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

TO THE GREATEST EXTENT PERMITTED BY LAW, THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. EXCEPT IN CASES WHERE IMPLIED WARRANTIES MAY NOT BE VALIDLY WAIVED, GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES AND COUNTRIES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE INSTANCES, GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.

The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

1. Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: www.grahamfield.com.
2. Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
3. Claims for any short shipment must be made within three (3) days of the invoice date.



Manufactured By:
GF Health Products, Inc.
1 Graham-Field Way, Atlanta, GA 30340
Made in USA

© 2026, GF Health Products, Inc. All Rights Reserved.
Basic American Medical Products is a trademark
of GF Health Products, Inc.

BA_FP3076-INS-LAB-RevA26



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

